



Instructions for use

Operating Chair
HP



Thank you for choosing BTC Srl products

Before using the BTC operating chairs, we kindly ask you to read this manual carefully and to conserve it for future reference: this will allow you to get the best out of the chair and to maintain its performance, safety and reliability over time.

INDEX	
PACKAGING AND TRANSPORTATION	3
TECHNICAL FEATURES	3
SYMBOLS USED	4
WARNINGS	4
ADJUSTING SEAT HEIGHT	5
ADJUSTING BACKREST ANGLE	5
ADJUSTING BACKREST HEIGHT	5
IDENTIFICATION	5
CLEANING AND DISINFECTION	6
NORMAL MAINTENANCE	6
DECLARATION OF COMPLIANCE	7



Packaging, transportation and assembly	
<p>Open the box at the top side, Take out the body of the operating chair, the base and the gas spring.</p> <p>Place the base on the ground and insert the gas spring into the tapered seat.</p> <p>Insert the body of the operating chair into the end of the gas spring.</p> <p>The operating chair is ready to use.</p>	

TECHNICAL FEATURES

Maximum load on the seat	180 kg
Maximal load applicable to the backrest	80 kg
Weight of the operating chair	14.5 kg
Adjustable inclination of the seat	0 -3°
Packaging dimensions	62x62x65

Dimensions of the operating chair



SYMBOLS USED	
WARNING, please consult the user manual before using	
Complies with Council Directive 93/42/CEE "MEDICAL DEVICES"	

WARNINGS
<p>Intended use: The HP chair is intended for use by a podiatrist and to guarantee correct posture and safe usage during outpatient care.</p> <p>Do not exceed the loads indicated in the section "Technical features". Do not remove the labels attached to the device. Should these become damaged, have them replaced.</p>
<p>BTC accepts no liability for any damage caused by failing to observe the warnings and instructions contained in this manual.</p>

<p>ADJUSTING SEAT HEIGHT</p>	
<p>Push upwards to adjust the height</p>	
<p>ADJUSTING BACKREST HEIGHT</p>	
<p>Press the button to adjust the height of the backrest.</p>	
<p>IDENTIFICATION</p>	
<p>The operating chair is identified by the rating plate that is located in the area indicated in the picture.</p> <p>If technical assistance is required or when ordering replacement parts, always indicate the serial number of the operating chair.</p>	

CLEANING AND DISINFECTION

To clean and disinfect the external surfaces of the operating chairs (part, painted and upholstered parts), use standard commercially available disinfectants, ensuring that the active substances listed are present in the maximum concentrations indicated.

WARNING!

Do not exceed the concentrations indicated: do not use products containing alcohol, ammonia, abrasive substances or benzol; wipe the disinfected parts with a cloth dampened with water (the disinfectants, even if diluted, can spoil the surfaces).

- Glutaldehyde 2%:10%
- Ethanol 96%:40%
- Formaldehyde:0,01%
- Glyoxal:0,15%
- Propanol:35%

The products are to be applied using a soft cloth dampened with disinfectants:do not spray the disinfectants directly onto the apparatus in order to avoid seepage into the apparatus, which could affect its performance.

NORMAL MAINTENANCE

The seat does not require special maintenance; periodically clean the wheels so that they slide better.



BTC Srl
Via Altobelli Bonetti, 8/A
40026 Imola (BO)
Tax code / VAT number (Italy): 02489471207
Administrative Economic Database (Italy): 443644
Share capital € 116,000 fully paid

Tel: +39 542 643664
Fax: +39 542 647391
<http://www.btc-med.it>
Email: btcmmed@btc-med.it

**DICHIARAZIONE DI CONFORMITA'
DECLARATION DE CONFORMITE
DECLARATION OF COMPLIANCE
ÜBEREINSTIMMUNGSERKLÄRUNG
DECLARACIÓN DE CONFORMIDAD**



BTC Srl dichiara che i dispositivi medici

BTC Srl declare que les dispositifs médicaux

BTC Srl declares that the medical devices

BTC Srl erklärt, dass die ärztlichen vorrichtungen

BTC Srl declara que los dispositivos médicos

Modello / Modèle / Model / Modell / Modelo	Matricola / No. de fabrication / Serial No. / Fabrikationsnr / Número de serie da/de/from/vom/de ÷ a/a/to/bis/a
Operating chair HP gel high without footrest	9120122-000000 ÷ 9120122-999999
Operating chair HP gel high with footrest	9120123-000000 ÷ 9120123-999999
Operating chair HP gel low without footrest	9120113-000000 ÷ 9120113-999999
Operating chair HP gel low with footrest	9120114-000000 ÷ 9120114-999999

sono conformi ai requisiti applicabili per i dispositivi di classe 1 della direttiva 93/42/CEE

sont conformes aux qualités requises applicables da la directrice 93/42/CEE

comply with the applicable requirements of Council Directive 93/42/CEE

mit den Anforderungen der Richtlinie 93/42/CEE

se conforma con los requisitos aplicables del directorio 93/42/CEE

Imola, 18/09/2019

BTC Srl